

DJMed

DJMED.COM.AU
PO BOX Q541
QUEEN VICTORIA BUILDING
NSW 1230

Prestige Mobility®

10cm PU Raiser
MPN 73801

WARRANTY STATEMENT

Take time to go through this manual which gives you more details about your Toilet Seat Raiser. Please keep this booklet handy and use it as and when required.

Toilet Seat Raiser is warrantied to be free of defects in material and workmanship for twelve months from the date of purchase. This device was built to exacting standards and carefully inspected prior to packing and shipping.

This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers. In the

event of a defect covered by this warranty, we will at our option supply parts or replace the unit.

This warranty does not cover device failure due to owner's misuse, poor installation or maintenance or negligence or normal wear and tear. Incorrect installation and maintenance may void warranty.

This warranty does not extend to nondurable components such as clips which are subject to normal wear and need periodic replacement.

INSTALATION

- Remove the existing toilet seat from the bowl. Seats are generally attached by a pair of bolts/nuts located under the rear rim of the toilet.
- Unscrew the side knobs until they extend out from the Raised Toilet Seat.
- Position the Toilet Seat Raiser onto your toilet bowl.
- **IMPORTANT: Front lip MUST be placed hanging over the front of the toilet and does not rest on the seat at all times.**
- Tighten the side knobs until the fixing is secure. Do not over tighten. Proceed to tighten hand-wheels until seat is secure and the brackets are snug against the bowl.
- Ensure Toilet Seat Raiser is firmly fixed before using. Hand-wheels are easily tightened so stop turning when resistance is felt. You should not be able to move the Raised Toilet Seat, once fitted.

REGULAR MAINTENANCE

- Check and make sure that the Toilet Seat Raiser has not shifter from the correct installed position before use.

REGULAR MAINTENANCE

- Regularly check the side knobs are tight and the toilet seat raiser is firmly clamped to the toilet bowl.
- The seat can be removed fully for thorough cleaning by unscrewing the side knobs.
- To clean the unit after each use, wipe clean as required with a bathroom disinfectant and a soft cloth.
- Do not use any abrasive cleaners or scourers that will scratch.
- Do not use water temperature in excess of 70 degrees Celsius.
- Before use inspect and confirm that the raiser has been installed correctly and is attached correctly and securely.

SAFETY

- If the toilet seat raiser is not properly assembled, installed and maintained injury and damage to the toilet seat raiser could result.
- Maximum load capacity must be observed.
- Always place even weight on the toilet seat raiser when moving on or off the toilet, to avoid the toilet seat raiser coming detached or tipping.
- The load capacity is not based on the lid of the toilet seat raiser. The lid is not suitable for use as a seat.
- The toilet seat raiser is not suitable for continuous use as a seat.
- Do not hold onto the seat as a support when getting on and off the toilet seat raiser.
- Under no circumstances should you stand on the raised toilet seat.
- Ensure that the side knobs are tight enough to hold the seat to the toilet bowl.