



Domus Auto Bariatric


Alternating Pressure Redistribution System

- Suitable for patients in medium to high risk of pressure injury
- Automatic adjustment providing optimal pressure according to patient's weight
- CPR knob for quick deflation & maximum security
- Heel relief function for optimal heel protection




Order Product Code: **9C077046**
Order Online: **dearjane.com.au**

APEX
Wellell Group

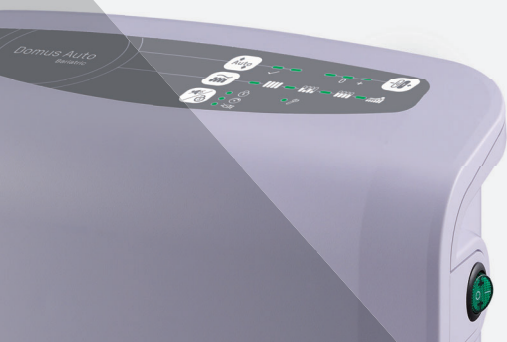


Domus Auto Bariatric

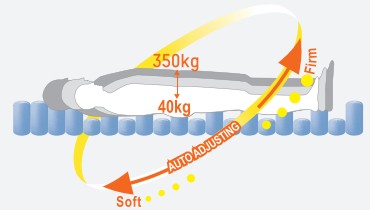

Alternating Pressure Redistribution System



Cell-on-Cell Design (8")
 The bottom cells prevent the patient from bottoming out in case of power failure

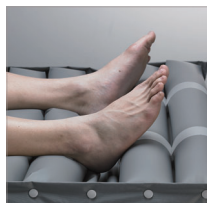

Heel Relief Function
 Individual detachable air cells prevent pressure injuries on the heels



Automatic Adjustment
 Automated calibration for constant pressure adjustment according to the patient's weight and position



Tuning
 Pressure tuning option to meet patient's comfort preference



Visual and Audible Alarm

- Low Pressure
- Power Failure
- Tech Support



CPR Knob
 Designed for quick mattress deflation in case of emergency and prevents accidental triggering



4-Way Stretch Cover
 Low friction and shear cover with high moisture-vapour transmission rate for better microclimate management



Biocompatibility




Vapour Permeable



Water Resistance



Flame Resistance

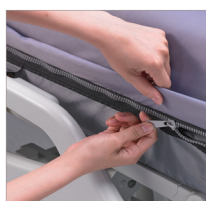


Enhanced Cross-Infection Control
 Made with all-around-zipper and welded seams for easy removal and prevents fluid ingress



Quick Connector with Cap
 Facilitates system installation and seals the mattress for patient transportation up to 8 hours

Pressure Relief During Alternating


Pressure mapping was conducted by placing a 180 kg test subject on the mattress in supine position under alternating mode. 100% of interface pressures during its 30-minute cycle are all below 32 mmHg.

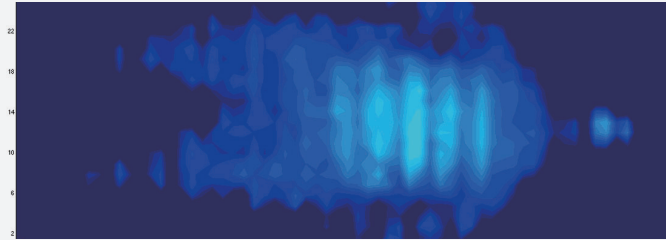

Order Product Code: **9C077046**
 Order Online: **dearjane.com.au**

5 Modes to Fulfill Individual Needs

- Max Firm
- Alternating
- Continuous Low Pressure (CLP)
- Seat Inflation w/ Alternating
- Seat Inflation w/ CLP

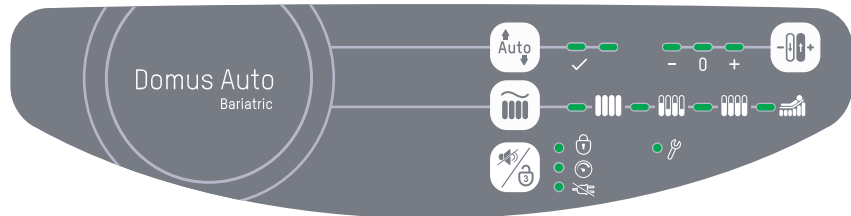

Alternating

- Alternately inflates and deflates air cells allowing pressure redistribution and encourage tissue reperfusion



Continuous Low Pressure (CLP)

- Provides a constant static surface in lower pressure with a greater contact area for patients to be immersed and enveloped better



Max Firm

- A static firm surface for nursing procedures. It will automatically return to alternating mode after 20 minutes



Seat Inflation

- Adjust the pressure to suit the patient in a sitting position. It can be used with alternating or CLP mode

Specifications

Model	Domus Auto Bariatric	
Pump	Dimension	29 x 18.5 x 12.6 cm
	Weight	2.2 kg
	Case Material	Flame Retardant ABS
	Supply Voltage	AC 220 – 240, 50 Hz
	Operating Cycle	10 mins
	Mattress	8" Replacement
Mattress	Dimension	200 x 120 x 20.3 cm
	Cell Height	8" x 20 Cells
	Weight	8 kg
	Cover Material	4-Way Stretch
	Cell Material	Nylon TPU
	Maximum Patient Weight	350 kg
	Flame Retardant Standards	EN 597-1, EN 597-2, BS 7175 Crib 5